LEATT®

X-FRAME HYBRID KNEE BRACE

**ADULT
S-XXL**


**GB|6 D|10 IT|14 FR|18 ES|22 BP|26
NL|30 SV|34 JP|38 RU|42 BP|46**

READ BEFORE USE

Fig.1

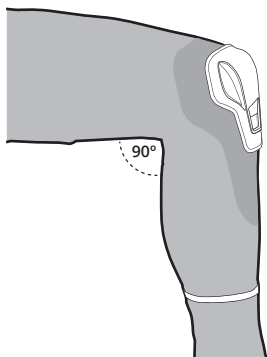1.1

1.2



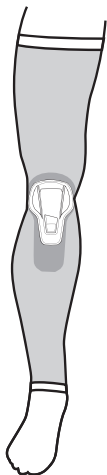
Fig.2

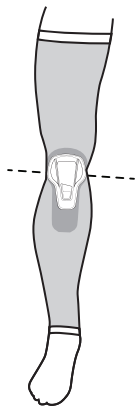2.1

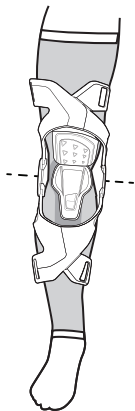2.2

2.3

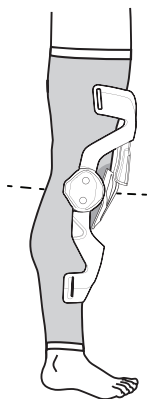
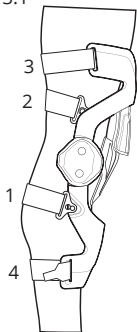
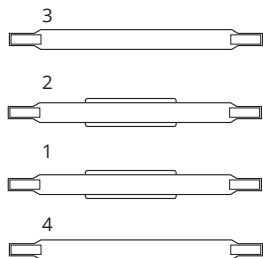
Fig.3

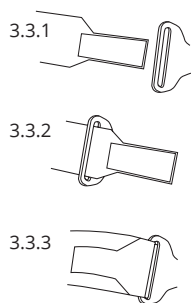
3.1


3.2


3.3

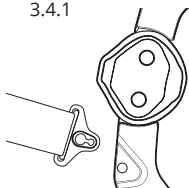
3.4.1

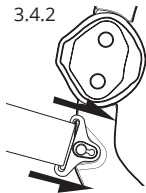
3.4.2

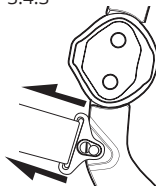3.4.3

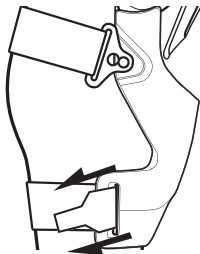3.5.1

3.5.2

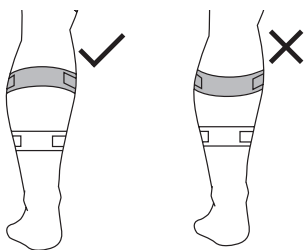
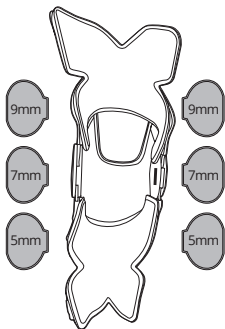
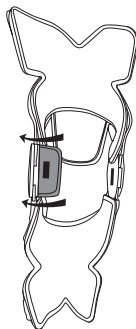
Fig.4

4.1

4.2

4.3


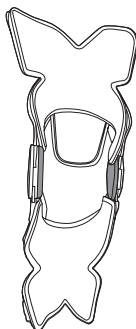
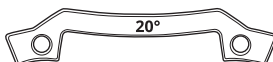
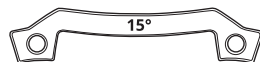
Fig.5

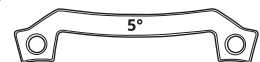5.1

5.2

0°

5°

10°

15°

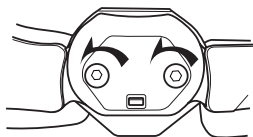
20°



5.3.1

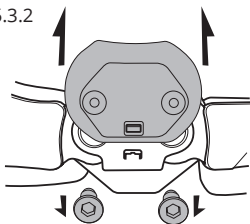5.3.2

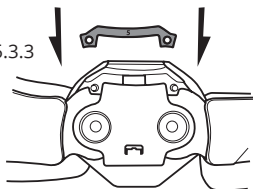5.3.3

5.3.4



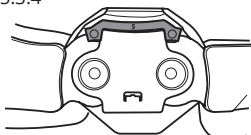
Fig.7

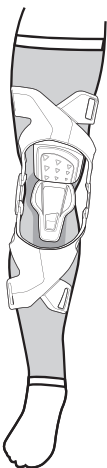
7.1

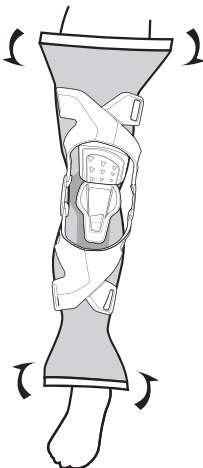7.2

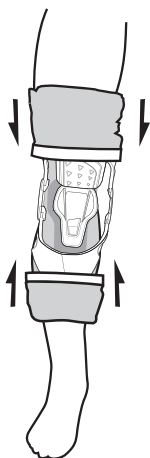7.3

 **WARNING**

Leatt® Corporation's personal protective equipment is not designed or intended to protect a wearer against serious bodily injuries caused by hard collisions or crashes. Protective equipment such as back or lumbar protectors do not protect against spinal injury. No protective equipment can protect the wearer from all foreseeable impacts.

**Fig.1 (p.2) - Sleeve Fitment:**

Fit the inner sleeve, provided with your brace, onto the leg. It is symmetrical, so it can be worn on the left or right leg. The hard shell knee cup should be centred over the knee, so the leg can bend freely.

Fig.2 (p.2) - Knee Brace Positioning**Fig.3 (p.3) - Strap Adjustment:**

Start with strap 1 and work your way through to strap 4, adjust and fit to a comfortable tolerance, making sure the fittings are correctly clipped into the body of the X Frame Brace.

When fitting the calf straps, always make sure the top strap is positioned above the calf to prevent the X Frame Brace from sliding down your leg.

Fig.4 (p.4) - Knee adjustment:

Additional spacers supplied to adjust pressure point inside knee.

Fig.5 (p.4) - Angle Adjustment:

The degree spacers provided allow you to change the angle of the hinge to prevent lock out and maintain a comfortable riding position. Make sure you use the same degree of spacer for the inner and outer hinge.


Fig.7 (p.5) - Sleeve Fold Over:

Once the X Frame Brace is in place and the fitment feels comfortable, fold the top and bottom of the sleeve over the brace making a secure fit.

This also decreases the chance of the pants catching on the X Frame Brace whilst riding.

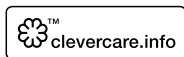Cleaning

Wipe only with a soft damp cloth in gentle soap or mild detergent in cold water, rinse by wiping with a soft cloth dampened in cold water and dry with a soft cloth. Comfort padding can be hand washed and air dried.

Common substances applied to this product (solvent, cleaners, hair tonics, etc.) can cause damage that may be invisible to the user and compromise the effectiveness/safety of your Leatt® product. If in any doubt, please contact the manufacturer before cleaning or laundering your protector. Always refer to the material content label on the Leatt® product. None of the materials used in the construction of the product are known to contain anything that is harmful or might cause an allergic reaction or be dangerous to health.

WARRANTY

Your Leatt® product is warranted to be free of defects in materials or manufacturing for one year from date of purchase. Your Leatt® product may break under certain conditions of use, so damage caused by accidents, falls, abuse or misuse is not covered by this limited warranty.

To make a warranty claim, go to www.leatt.com or send an email to warranty@leatt.com.

LEATT® CORPORATION EXPRESSLY DISCLAIMS ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

Contact details

Head Office:

No 12, Kiepersol Crescent, Atlas Gardens,
Contermanskloof, Durbanville, 7550,
Cape Town, South Africa
Tel: +27 21 557 7257
Fax: +27 21 557 7381
Email: info@leatt.com
Website: www.leatt.com

Postal Address:

Suite 109, Private Bag X3, Bloubergrant,
7443, South Africa

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A series of 20 horizontal dotted lines spanning the width of the page, providing a guide for handwriting practice.

- Injected Carbon Composite Frame
- Low profile knee cup to fit comfortably inside most pants
- Hybrid AirFlex impact gel knee cup with hard shell slider for ultimate knee brace comfort
- Asymmetrical Hinges
 - 40% slimmer inner hinge for superior bike feel
 - Metal geared outer hinge for precise movement and durability
- Reduction of forces to limit knee injuries: ACL, Meniscus and MCL
- Certified as medical device: EU CA014741 | USA FDA 10048761
- CE tested and certified as impact protection: Knee EN1621-1
- Hyper extension limitations 5°, 10°, 15° and 20° for ACL injury reduction
- Fixed hyper flexion stoppers for meniscus injury reduction
- Low-profile shin bone pad that fits inside all boots
- Comfortable and easy fitting strap system
- Customized fit with interchangeable hinge padding sizes
- Size: S-XXL

CARBON
CONSTRUCTION

AIR-FLEX

BARCODE

LEATT.COM

Made in China | Leatt® 103-20